

May 26, 2015

**URGENT MEDICAL DEVICE NOTIFICATION**

**Important Information For Patients With Animas<sup>®</sup> 2020 and/or IR 1200 Insulin Pump(s)**

At Animas, we hold our products to the highest standards of quality and are committed to communicating any issues impacting the operation of Animas products.

In December 2012, we notified patients of an end of use date issue that will affect the Animas<sup>®</sup> 2020 and IR 1200 insulin pumps as follows:

- **Animas<sup>®</sup> 2020 insulin pump:** The pump will only operate until midnight on December 31, 2015. After this date, the pump will no longer deliver insulin and will generate a “Call Service” Alarm. This end of use date was not originally included in the product labelling.
- **IR 1200 insulin pump:** After midnight on December 31, 2015, the pump’s calendar will revert to a previous year. You will not be able to set the correct date as the pump’s calendar will not recognize the year 2016 or beyond. Although the pump will continue to deliver insulin, if you are using a data management software program with your pump, you will notice inaccuracies in the reports because of the incorrect date. This end of use date was not originally included in the product labelling.

Please be assured that this issue does not affect the operation of these pumps prior to January 1, 2016.

**We are requesting that all patients with an Animas<sup>®</sup> 2020 and/or IR 1200 insulin pump(s) contact Animas Canada at 1-866-406-4844 as soon as possible to arrange for the return of their pump to Animas and/or confirm their transition to a newer pump model.**

Should you have any questions or concerns about this notice or if you no longer use or have your Animas<sup>®</sup> 2020 and/or IR 1200 Insulin Pump, please contact Animas Corporation at **1-866-545-3831**.

Thank you for your cooperation and support.

Sincerely,

Animas Corporation